MANUAL DE INSTRUCCIONES / INSTRUCTION MANUAL

Andador/ Rollator VICTORIA, CALETA, ALHAMBRA

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 **MOBICLINIC**
Spanish Leading Company Since 1985

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ENGLISH

Please read carefully these instructions to know your new rollator well before use. A rollator is used by people who need support while walking due to unsteadiness, disability or recovery from surgery. These models have a weight capacity of 100kg.

HOW TO USE:

1. To activate the brakes, press up on the handlebars.
2. Place both feet side-by-side in the middle of the rollator frame.
3. Release the brakes by pulling the brake handles up.
4. Grasp the hand grips on either side of the rollator.
5. Push the rollator forward to a comfortable distance. Y
6. Step forward toward the inside of the rollator frame with your weaker leg.
7. Step forward with your stronger leg while supporting yourself with the hand grips. Both feet should be side-by-side between the wheels of the rollator. Continue walking by repeating Step 5 and Step 7.

TIPS AND WARNINGS:

- Always keep a part of your body, such as your toes, inside the rollator frame.
- When standing up from a seated position, push yourself up from the seat rather than pulling on the handlebars of the rollator.
- Take short steps to turn and keep your hips parallel to the front bar of the rollator. Avoid turning your body away from the rollator.
- The rollator must be braked if you are seated. Do not use it as a wheelchair. It can only be used to rest temporarily while being braked.
- Do not apply force on one side only or the rollator will tip over.
- Do not go up or down stairs or steps with the rollator.
- Do not adjust the rollator during use. Assembly and disassembly of the rollator should be done only when the rollator is not in use.
- Do not walk on escalators with a rollator.
- Do not change the direction of the rollator suddenly when using the rollator on a slope.
- It must be used on level streets. Do not use on bumpy roads. Avoid

hitting the rollator against obstacles.

- Do not walk with a rollator uphill with an incline of more than 7 degrees. Do not walk a rollator downhill with a slope of more than 15 degrees. When walking uphill or downhill, the lateral inclination or slope gradient should not exceed 3.5 degrees.
- Clean the rollator regularly with a clean, dry cloth.

NOTICE:

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

INDICATIONS:

- It is designed to assist walking for the old, the disabled and the people who has walking problem.
- Make sure that all parts are tightened properly and all four wheels are touching the floor before using it.
- Weight capacity 100kgs.

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**GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD**
SUNGO EUROPE B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.