# MANUAL DE INSTRUCCIONES

## Silla con inodoro BARCO

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SWE
Instrukcja obsługi	PL
Brugsanvisning	DK

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## NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

## 1. CLEANING AND MAINTENANCE

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth. Do not use solvents, bleach, synthetic detergents, wax polishes, aerosols or other abrasives that may damage your product.

## 2. INDICATIONS

- It is designed for the elderly, patients and the disabled, as a commode chair.
- Weight capacity 100 kg.
- Check and make sure that all parts are tight and locked before using the chair.
- Make sure the wheels are locked before sitting in the chair.

## 3. TIPS AND PRECAUTIONS

- Must be used indoors on flat surfaces. Do not drive it on a bumpy road. Avoid hitting the wheelchair against obstacles.
- Do not drive the wheelchair on elevated areas with the force of inertia. This could damage the wheelchair or cause an accident.
- Do not go up or down steps and do not climb stairs without the help of an assistant. Do not lift the wheelchair with the user seated in it, otherwise it will damage the wheelchair and cause injury to the user.
- Unless specified, do not adjust, assemble or disassemble the chair with the user seated. Only adjust, assemble or disassemble the chair when it is not occupied.
- Do not lean your body out of the chair to prevent the chair from tipping over.
- Do not change the seated position when moving backwards, otherwise it may cause the chair to tip over.
- Do not drive the wheelchair on escalators.
- Do not step on the footrest when getting in and out of the wheelchair. Do not stand on the footrest, otherwise it may tip over.
- Do not tilt the wheelchair or change direction suddenly. Do not brake the wheelchair suddenly.

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**GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD**
**EC REP** **SUNGO EUROPE B.V.**
**GRUPO R. QUERALTÓ S.A.**